

Test Kit Contents



Viral Transport
Media Tube



Nasal Swab



Biohazard Bag

WARNING

- Do not spill the liquid in the sample tube.
- Do not drink or breathe in the liquid in the sample tube.
- Do not let the liquid in the sample tube touch your skin or eyes.

1



Sanitize your hands prior to beginning sample collection. With clean hands, unscrew the collection tube. Place the cap face up on a flat surface, making sure it does not get contaminated.

2



Open the nasal swab package by pulling apart the wrapping. Pull the swab out by its end, making sure not to contaminate the tip of the cotton swab.

3



Insert the nasal swab into your nostril, about one inch (~ 2 cm) until resistance is met. Gently rotate the swab several times, for ~10-15 seconds. Repeat in other nostril using the same swab.

4



Insert nasal swab into the collection tube liquid with cotton tip facing down.

5



Screw the collection tube cap back on. Make sure to tightly screw the cap to prevent any leaks.

6



Place your sample tube into the biohazard bag provided. Place sample in the appropriate collection bin.