


### COVID-19 Saliva Test Kit Contents



Tube



Funnel

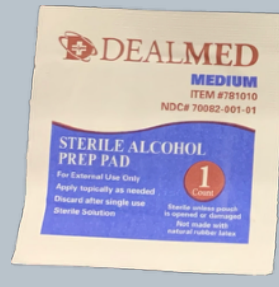

Alcohol Pad



Biohazard Bag

### IMPORTANT

- Do not have food, colored drinks, or mouthwash 20 minutes before collection.
- Drink water up to 20 minutes prior to collection to ensure adequate hydration.

1



Wash your hands for 20 seconds with soap and water prior to beginning sample collection. With clean hands, unscrew the collection tube. Place the cap face up on a flat surface, making sure it does not get contaminated.

2



Place the funnel inside the tube. You can hold the sides of the funnel to stabilize it if needed.

3



Gather saliva in your mouth. With the funnel inserted halfway into the tube, spit the saliva into the funnel. It may take a few seconds for your saliva to reach the tube. Continue until the tube is filled halfway. You may lift and lower the funnel to release saliva into the tube.

4



Screw the collection cap back onto the tube. Make sure the cap fits tightly back onto the tube. Clean the collection tube with the alcohol swab provided.

5



Place your sample tube into the biohazard bag provided.